



Dental impression kit instructions

- 1) Brush your teeth and write your name on the bag
- 2) Choose the impression tray to your size (your back teeth should not touch the edges)
- 3) Mix all of the catalyst paste with all of the base paste actively in your hands for up to 30 seconds. (until a uniform color is obtained)
- 4) Roll the dough obtained and place it quickly at the bottom of the impression tray as below



- 5) Place everything in the mouth and bite firmly with the upper teeth until the plastic
- 6) Put your upper lip over the plastic and press your cheeks and lips to make the dough rise as high as possible on your gums



- 7) Wait 1 minute 30 without moving before unmolding
- 8) Fill a glass/bowl with lukewarm water and soak the red wax arch in for 30 seconds as below :
- 9) Place the arch between your teeth and bite firmly on it as hard as possible



- 10) It's time to send us your photo impression on Facebook, Instagram, by text or by email for validation (contact@mg-protect.fr or +33 6.58.01.85.14)
- 11) If we validate it, leave it on the support, and put it back in the bag with the two unused pastes. Otherwise we will ask you to make a second try with the two remaining pastes
- 12) Finally, return the parcel and the unused material to the following address in a bubble envelope found in your post office :

SAS MG PROTECT
218 CHEMIN DE LA BANE
84450 SAINT SATURNIN LES AVIGNON, FRANCE